


C³

Flexible Penile Clamps

The C³ is a flexible penile clamp that protects against urine leakage in cases of mild-to-moderate incontinence.

Most penile clamps are rigid. As a result, they can restrict blood flow to the penis and are more easily noticed under clothes. The C³ conforms to your body and only puts pressure where it is needed.

The C³ is made of a soft foam that's held in place with an adjustable Velcro strap. There's a bump in the foam that puts gentle pressure on your urethra. Compressing the urethra is the key to stopping urine leakage. Because this bump is only over the urethra, the C³ doesn't block blood flow to your penis, and that's the key to its comfort and its safety.

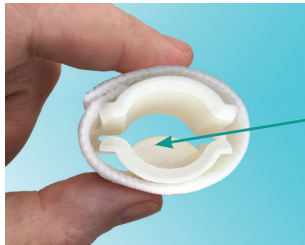

It's the
Bump!

Important Information

- Urinate at least every four (4) hours when you use the C³.
- To urinate, release the strap, open the device and flatten it against your body - then urinate as usual.
- Recommend use of this device is as needed during the day. Use a pad at night.
- Do not use the C³ if you have a urinary catheter or penile prosthesis.

Frequently Asked Questions

Even though the C³ is an OTC device, should I still see a doctor?

Yes, it's a good idea. Urine leakage is a symptom, so you should see your doctor to find out what's causing you to leak.

Does one C³ size fit all?

No. There are three sizes available. The regular size fits most men.

Does C³ restrict blood flow in the penis?

No. The C³'s unique shape is designed to stop just the flow of urine.

Can I clean or wash the C³?

No. If you wash it, the C³ may not work effectively. A device that becomes soiled, wet or damaged should be replaced.



For information
and to re-order:

8531 154th Ave NE #150
Redmond, WA 98052
866-839-9260
425-497-1045 Fax
www.personalmed.com

© 2007-2022 Personal Medical Corp. All rights reserved.

C³ is a registered trademark of Personal Medical Corp.

USPN 5,184,629 with global counterparts.

290046-005 Rev P January 2022

C³®

Flexible Penile Clamps



*Comfort, Control
and Confidence*





Flexible Penile Clamps

The C³ is Easy to Use

Here's how it works:

1. Unfasten the strap and place your penis through the opening.



The bump should be underneath your penis and the hinges should face **away** from your body.

2. Close the device by bending it towards you so it clamps around your penis.
3. Position the device slightly away from your body and tighten the strap.



When the C³ is properly positioned, its bump will block the flow of urine by pressing against your urethra (located in the middle of the underside of your penis).

To urinate or to remove the C³, loosen its strap and completely unclamp the device.

Choose the Size that's Right for You

The C³ device comes in three sizes (Small, Regular and Large). Sizes refer to circumference, not length.

Getting the right size is important for effectiveness and comfort, so please follow these instructions to determine the size that's right for you:

1. Use the ruler at right or a flexible tape measure to measure yourself.
2. Measure only a non-erect penis. For best results, conduct sizing when your body is at room temperature. For example, don't conduct the sizing right after a shower.
3. Wrap the ruler comfortably **around** your penis, slightly away from the body. (Do not measure the length.)
4. When the ruler is snug around your penis, read the number in inches to determine your size before ordering:

Small = less than 2½ inches around
 Regular = 2½-4 inches around
 Large = 4-5½ inches around

Please call us if you have any questions or need help.



C³ Order Form

Please complete this form and mail to:

Personal Med
 8531 154th Ave NE #150
 Redmond, WA 98052

Or call to place your order: 866-839-9260

Name _____

Address _____

City _____

State _____ Zip _____

Telephone _____

Bags of 6 ea. _____ @ \$60.00 _____

Small Regular Large

7-10 Day Shipping Free

Priority Shipping \$10.00

Sales Tax (WA only) _____

Order Total

The C³ Urinary Control Device is sent to you in discreet packaging.

Method of Payment - Check or Credit Card (No cash or HSA cards):

Visa MasterCard AmEx Discover

Card Number _____ Exp. Date _____

Signature _____ CVV No. _____

Questions? Call 866-839-9260



Set printer to Actual Size.
Cut out this ruler to measure AROUND your penis.